
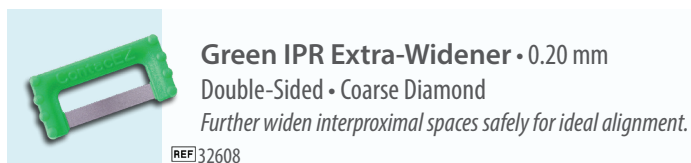





The ContacEZ IPR Strip System is a safe and innovative interproximal reduction system that makes interproximal enamel stripping efficient and accurate without creating sharp corners or subgingival ledges. Designed for use with Invisalign[®], ClearCorrect[™], eClinger, Six Month Smiles[®], and other orthodontic treatments, these flexible diamond strips curve and conform along the natural contours of the teeth to maximize patient comfort and safety.

- ▶ Ergonomic, single-handed design
- ▶ Provides optimal tactile control
- ▶ Will not remove excessive enamel
- ▶ Will not cut lips or gums
- ▶ Prevents sharp corners or subgingival ledges
- ▶ Effective for anterior and posterior areas
- ▶ Central opening for maximum visual perception
- ▶ Saves chair-time

ContacEZ IPR Strip System

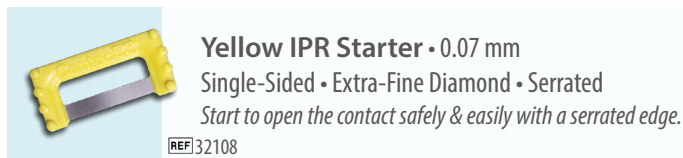

ContacEZ IPR Optional Strips

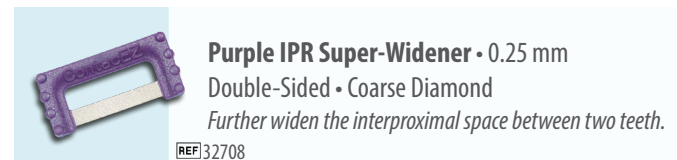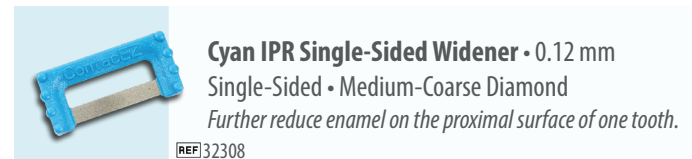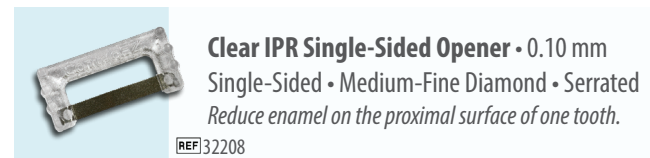

Usage Guidelines & Technique

This technique is especially effective for anterior teeth and for case refinement. ContacEZ IPR Strips are autoclavable (**steam heat only, up to 270°F for 10 minutes at 30 psi max**) and reusable, and should be sterilized prior to the first use. **The manufacturer recommends to use each strip 3-5 times for maximum efficiency and functionality.**

- ① Hold the strip between your thumb and forefinger, insert the strip into the interproximal space, and gently pass the strip back and forth buccolingually until the strip is passive. Do not use vertical force. Start with the thinnest diamond strip, and repeat with thicker strips until you have reduced (stripped) the correct amount of enamel according to your treatment case prescription.
- ② To ensure accurate measurements, polish the proximal surface using the ContacEZ *Gray Final Polishing Strip*.
- ③ Verify the width of the gap created using the ContacEZ *Incremental Thickness Gauge*. The location and amount of any reproximation performed should be recorded in the patient's chart.
- ④ Pediatric dentists can use ContacEZ IPR Strips for the serial proximal reduction of primary molars.



Patented
contacez.com

100% Guaranteed
360.694.1000



EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands



ContacEZ, LLC
217 SE 136th Ave, Suite 105
Vancouver, WA 98684
United States