



STEP 1

Place a cotton roll between the lip and gum. Insert the strip into interproximal spaces by passing back and forth buccolingually with minimal vertical pressure.

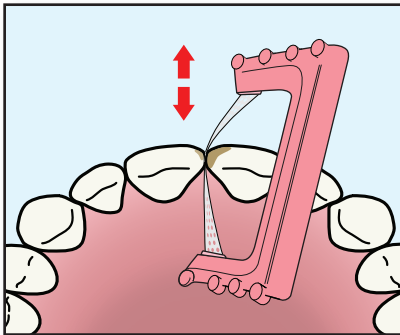

STEP 2

Use gentle force to curve the flexible strip along the stained interproximal surface. Pass the strip back and forth buccolingually, maintaining the pressure on the stained surface until the discoloration has been removed.

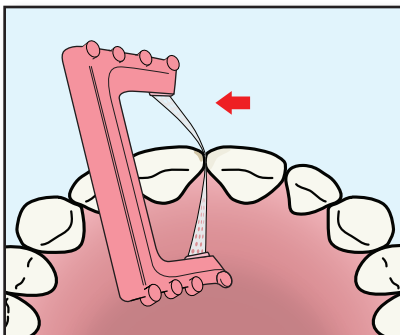

STEP 3

Repeat as necessary on the opposite tooth. The strip will curve and clean along the natural curvature of the tooth with its built-in flexibility. The non-abrasive perforated strip will not damage enamel, just remove the stain.



Protect the lip by using either a cotton roll or fingers to move it out of the way. Insert the strip with minimal vertical pressure.



Passing the thin, perforated strip back and forth buccolingually will even remove stains between tight contacts.



Patented

contacez.com

100% Guaranteed

360.694.1000



EC

REP

EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands



ContactEZ, LLC
217 SE 136th Ave, Suite 105
Vancouver, WA 98684
United States